




NOMI TANG



IntiMate designed for women who want to enhance their sensuality

IntiMate works to strengthen the entire pelvic floor and makes it more responsive to touch, improves muscular control and allows for the most intense and stimulating orgasms. Made from skin-friendly materials used in many medical applications, it is shaped ergonomically and it is completely waterproof.

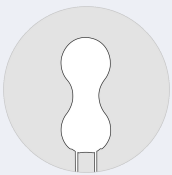


DIFFERENT TRAINING STAGES

The different weights of the balls are used for various training stages. The quality of the balls is strictly tested and controlled in order to allow a safe experience and the best training results.



MORE EFFECTIVE THAN OTHER KEGEL BALLS

The appearance adopts a petal-shaped design, which is aesthetic and ergonomic. They special weights not only help strengthen the muscles but also provide a superb pleasure experience during exercise.

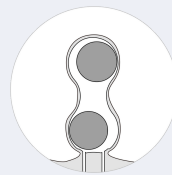

ERGONOMICALLY SHAPED

The balls continuously collide with the inner wall of the vagina during exercise, which stimulates a physiological reaction and a natural contraction which results in a better vaginal tightening.



SOFT AND COMFORTABLE SILICONE

The product features a very soft and comfortable medical-grade silicone material. The balls adapt any angle with the movement during exercise to obtain the best tightening effect.



NOMI TANG

IntiMate

HOW TO USE

- Start with the light ball and change to the heavy ball to increase the level of training once you feel prepared to hold it comfortably.
- Choose a relaxed position. Slowly place the ball into the vagina (we recommend to apply water-based lubricant) and place it at least 2 cm away from the vaginal opening to ensure that the bottom ball is completely pushed over the pelvic floor muscle.
- During abdominal breathing training, squeeze 5 seconds and relax for 5 seconds to complete a cycle. It is recommended to exercise between 10 and 15 minutes, slowly increasing the time as the body adapts.
- After the exercise, please remove the ball slowly.

SPECIFICATION

Material:

FDA grade silicone, TPE and PC

Size:

Single (length / diameter):
40 / 37 mm

Double (length / diameter 1 /
diameter 2):
70 / 31 / 30 mm

Net Weight:

Single: 53g
Double: 107g

Gross Weight:

303g

Contents:

IntiMate, user manual

Warranty:

1 year

FEATURES

- Soft medical grade silicone
- Suitable for different training stages
- Ergonomic design for optimal fit
- Ball movement design for best effects
- Exciting pleasure while exercising
- Set of two Kegel balls

SHIPPING DETAILS

Packaging Size:

115mm x 115mm x 50mm


Shipping Size:

310mm x 370mm x 315mm

Total units per box: 24

Colour: Red Violet

EAN: 4897028220185

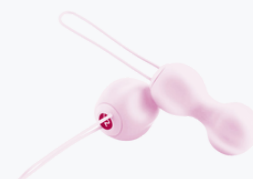

Colour: Sakura

EAN: 4897028220192



NOMI TANG

IntiMate Plus

HOW TO USE

- Start with the light ball and change to the heavy ball to increase the level of training once you feel prepared to hold it comfortably.
- Choose a relaxed position. Slowly place the ball into the vagina (we recommend to apply water-based lubricant) and place it at least 2 cm away from the vaginal opening to ensure that the bottom ball is completely pushed over the pelvic floor muscle.
- During abdominal breathing training, squeeze 5 seconds and relax for 5 seconds to complete a cycle. It is recommended to exercise between 10 and 15 minutes, slowly increasing the time as the body adapts.
- After the exercise, please remove the ball slowly.

SPECIFICATION

Material:

FDA grade silicone, TPE and PC

Size:

Double (length / diameter 1 / diameter 2): 70 / 31 / 30 mm

Net Weight:

Light: 83g

Double: 139g

Net Weight:

367g

Contents:

IntiMate Plus, user manual

Warranty:

1 year

FEATURES

- Soft medical grade silicone
- Suitable for different training stages
- Ergonomic design for optimal fit
- Ball movement design for best effects
- Exciting pleasure while exercising
- Set of two Kegel balls

SHIPPING DETAILS

Packaging Size:

115mm x 115mm x 55mm

Shipping Size:

310mm x 370mm x 315mm

Total units per box: 24

Colour: Red Violet

4897028220420



Colour: Sakura

4897028220437